


How to Make DIY AED Interface Cables for Delta 1600 AED Analyzer

It is simple and easy to make AED Interface cables.


1. Cut the wires that connect to the AED pads as shown in the photo to the right.

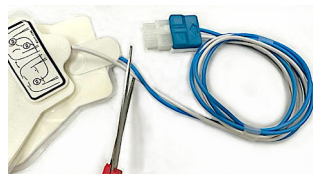

2. Strip the insulation about 0.4" off the end of the leads as shown.



3. As shown in the illustration and photo, connect the AED cable to Netech's open ended Interface cable PN# 610-6383.

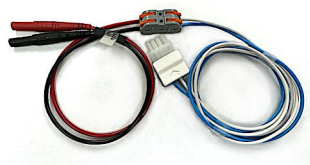

**Netech offers a variety of custom cables for most popular AED models.
Please contact us for all your custom-built AED cables.**

Contact Us Today 1-800-547-6557 sales@NetechBiomedical.com

Visit Us www.NetechBiomedical.com

